
ClearView “Sub-Q” Needle Set

Norfolk  Medical

7350 N. Ridgeway, Skokie, IL 60076 USA
tel. 847-674-7075 · fax 847-674-7066 · info@norfolkmedical.com

CAUTION: Federal (USA) Law restricts this device to sale by or on the order of a physician.

The enclosed needle infusion sets feature unique soft, round disks for increased patient comfort when the needle set is dressed. Read instructions prior to use.

Indications

Intended for subcutaneous infusion of fluid medicines.

Warnings

- Intended for **single patient use. DO NOT REUSE or RESTERILIZE.**
- Set contains small parts, which may present a choking hazard.
- Do not use of package is open or damaged.
- Set not intended for administration of medication into intravascular space.
- After use, this product may be a potential biohazard. Handle and discard in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Notes


- Product does not contain DEHP
- Product is not made with natural rubber latex

Instructions for Use

1. Use aseptic technique to prepare infusion set. Wash hands prior to use.
2. Remove the set from the sterile packaging. Remove the protector cap from the infusion set.
3. Attach and secure set to administration tubing or syringe and fill entire set with medication.
4. Select area of skin for insertion of the infusion set as directed by your physician. Clean the skin with antiseptic solution and allow to dry.
5. Hold the needle by grasping the edges of the disk and insert straight down into subcutaneous tissue.



-or-

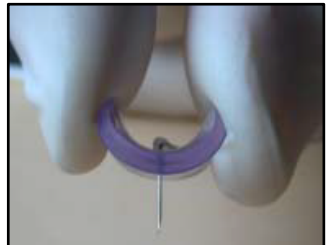


ClearView “Sub-Q” Needle Set

Norfolk  Medical

7350 N. Ridgeway, Skokie, IL 60076 USA
tel. 847-674-7075 · fax 847-674-7066 · info@norfolkmedical.com

CAUTION: Federal (USA) Law restricts this device to sale by or on the order of a physician.

- Place the disk flat against the skin, the needle will reside in the subcutaneous tissue. Check needle placement with a syringe by pulling the plunger back to check for blood return at the needle site. If blood is seen, remove the infusion set, discard in appropriate disposal container, and repeat steps 1-6.
- Cover and secure needle with provided transparent dressing.

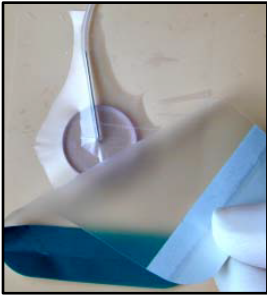

- Start infusion as directed per the medication and infusion delivery system.
- When infusion is complete, loosen the dressing, grasp the edges of the disk, and pull the needle straight up and out away from skin.
- Discard set and any loose components in appropriate disposal container.
- Apply pressure to the site with gauze and cover site with appropriate bandage if necessary.

NORFOLK MEDICAL DISCLAIMS ALL WARRANTIES, WHETHER EXPRESSED OR IMPLIED, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND NORFOLK MEDICAL WILL NOT BE LIABLE TO PURCHASER FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES ARISING OUT OF USE OR HANDLING OF THIS PRODUCT.



Do not reuse



Sterilized with EO



Manufacturer



Keep Dry



Product Code



Lot Code



Latex Free




Use by Date



Norfolk Medical Products, Inc.

7350 N. Ridgeway,
Skokie, IL 60076 USA

CE
0297

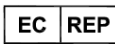

PHS Medical GmbH

Ederweg 3, 34277
Fuldabruck, Germany